



# Emergency Child Restraint

## USER INSTRUCTIONS

### PACK INCLUDES

3 x HARNESSSES

SMALL

4-18lb  
2-8kg

MEDIUM

13-46lb  
6-21kg

LARGE

42-110lb  
19-50kg

4 x UNIVERSAL COT  
RESTRAINT STRAPS




The Emergency Child Restraint is designed for use only in an emergency setting and only by suitably trained personnel. The Emergency Child Restraint is for professional use by one or more trained operators. The Emergency Child Restraint is not designed as an immobilization device and should not be used to immobilize the patient, or as part of an immobilization system.




SEE OVERLEAF FOR  
FITTING INSTRUCTIONS


## FITTING INSTRUCTIONS

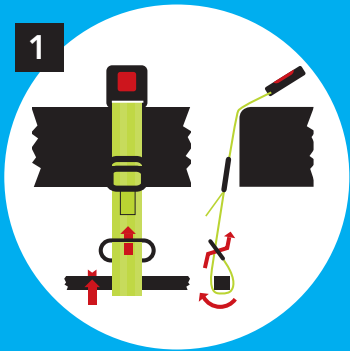


1 Attach straps 2 per side of the cot frame runner rail, pull buckle through metal ring

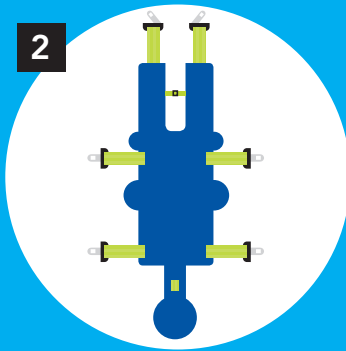


2 Position correct sized harness and spread it flat on the cot mattress with straps extended

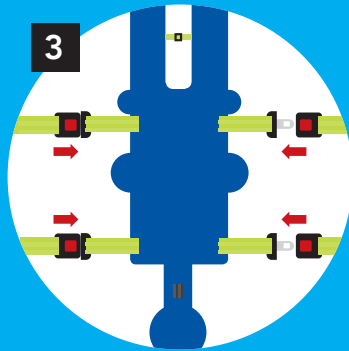


3 Using the 4 belt buckles attached to the cot frame, click in to place and secure to the supplied straps

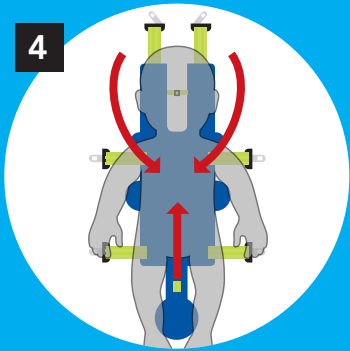

4 Once child is positioned bring shoulder straps over torso and crotch buckle between legs to meet straps

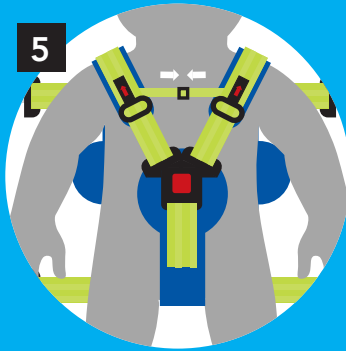

5 Engage shoulder straps into crotch buckle and tug shoulder tabs to tighten, click in chest strap

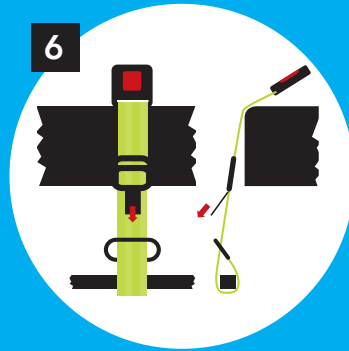

6 Once child is secured in the harness pull tabs on cot straps to secure harness to the cot mattress

## Maintenance instructions

The Emergency Child Restraint is comprised of exceptionally robust materials. Typical maintenance is not required. The full system should be inspected quarterly to ensure peak performance.

### Disinfection

The Emergency Child Restraint should be disinfected according to the protocols used to disinfect the cot mattress. Do not immerse buckles in fluid.

### Straps/All

Ambient temperatures may influence the tightening of the Bio-Coated straps. This is perfectly normal.

<https://emergencychildrestraint.com/watch-video/>

### Infant Comfort Care Head Block

Designed for infant care to reduce head vibration.

<https://emergencychildrestraint.com/watch-video/>



# Emergency Child Restraint

EMS SOLUTIONS LLC 263 MINEOLA BLVD,  
MINEOLA, NEW YORK 11501, USA  
[info@ems-sol.com](mailto:info@ems-sol.com)