

# BINDER LIFT™ Cleaning Instructions

## Nylon model

- The Nylon Binder Lift may be hand or machined washed.
- Remove BioThane leg straps before machine wash. Hand clean leg straps w/soap and water or germicidal disinfectant cleaner. Air dry.
- Connect all three horizontal compression straps prior to machine wash.
- Pretreat soiled areas with commercial spot and stain removal cleaner.
- Wash in warm soap and water. No bleach. Rinse thoroughly.
- Hang or tumble dry on medium heat settings. No HIGH heat.
- Do not dry clean.

## Vinyl model

- Hand wash ONLY. No machine wash or dry.
- Rinse off any bodily fluids with water.
- Wash Binder Lift with hot soap and water using soft brush or cloth.
- Max temp 175°
- Rinse thoroughly and hang/air or wipe dry.
- Disinfect with germicidal cleaning wipes or 2%-5% chlorine/water solution
- Alternative cleaning may be accomplished with standard hospital grade germicidal disinfectant cleaners such as AHP (accelerated hydrogen peroxide) or Sani Wipes