

BINDER TRANSFER INSTRUCTIONS FOR USE

INTENDED USE

The Binder Transfer is designed to assist emergency care providers with the physical movement and transfer of a patient from one surface to another (e.g., from a prehospital stretcher to a hospital bed). It is designed to be secured to the stretcher’s frame and remain in place under the patient until providers are ready to perform the transfer. Once ready, the providers simply need to release the buckles on the outer edge of the device and gently glide the patient over to the other surface.

BINDER TRANSFER



COMPONENTS

The Binder Transfer Starter Bundle includes the following pieces:

- (1) Binder Transfer slide sheet (Fig.1)
- (8) Binder Transfer anchors (Fig. 2)

MATERIALS

Slide sheet: 100% Nylon

Anchors: 100% PVC coated polyester

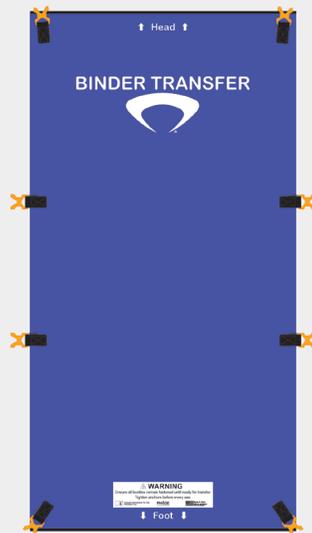


Fig. 1

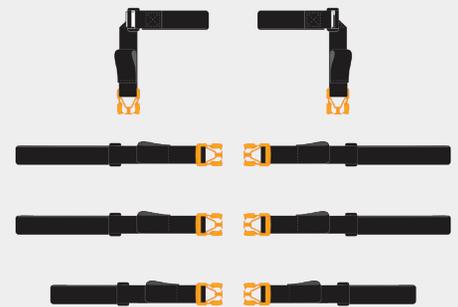


Fig. 2




Your feedback is essential in helping us improve. Please share your feedback by scanning the QR code and completing the survey.

**CALL OR EMAIL US WITH YOUR QUESTIONS
AND COMMENTS. WE'RE HERE TO HELP**


855-239-5438 | info@phaseintl.com | PHASEINTL.COM

ANCHOR INSTALLATION


1. Refer to the Binder Transfer Anchor Placement Guide to determine the recommended anchor attachment points specific to your stretcher's make and model.

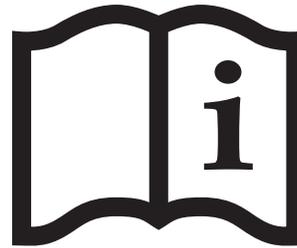

2. Refer to the label sewn under the plastic loop on each anchor to determine its proper placement on the stretcher.

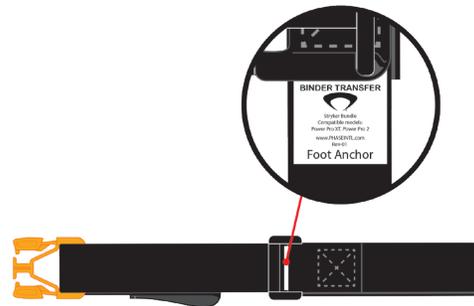

3. Secure each anchor to the stretcher's frame by threading the distal end under the stretcher frame and back up through the plastic loop. Then, bring the distal end of the anchor back onto itself and fasten the hook-and-loop materials together.

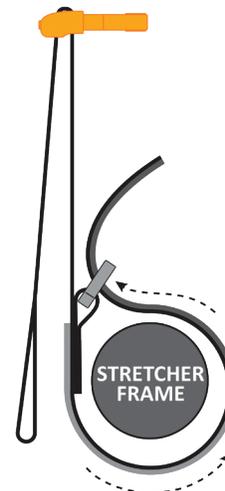

NOTE: Removing the stretcher mattress makes it easier to secure the Binder Transfer anchors.

SLIDE SHEET INSTALLATION

1. Begin by raising the stretcher backrest to the Semi-Fowler position.




2. Lay the Binder Transfer slide sheet on the stretcher mattress with the label facing up, aligning the head end just below the stretcher's headrest.




SLIDE SHEET INSTALLATION

3. Beginning with the middle anchors, fasten all anchors to the slide sheet and gently tighten.

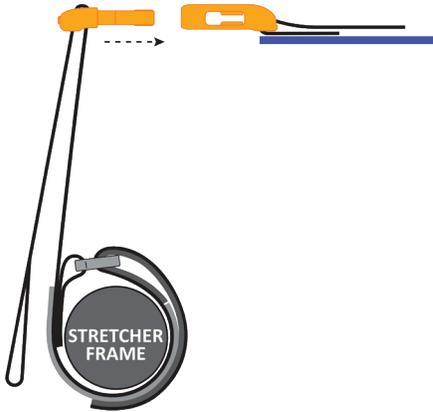

4. Once all anchors are fastened and the Binder Transfer slide sheet is evenly spaced across the mattress, re-tighten anchors until the slide sheet is pulled tight.

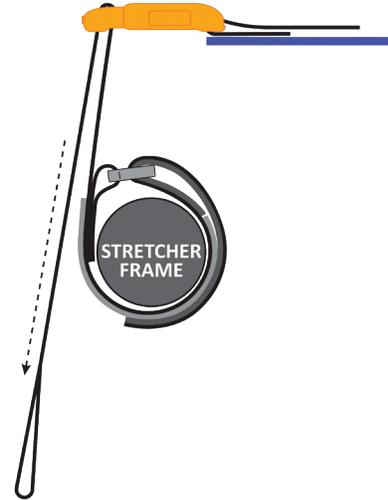


5. Congratulations! You are now ready to put your stretcher into service.




BINDER TRANSFER DEPLOYMENT

1. Once you are ready to transfer a patient, prepare both surfaces for patient transfer according to your department's protocols and guidelines.

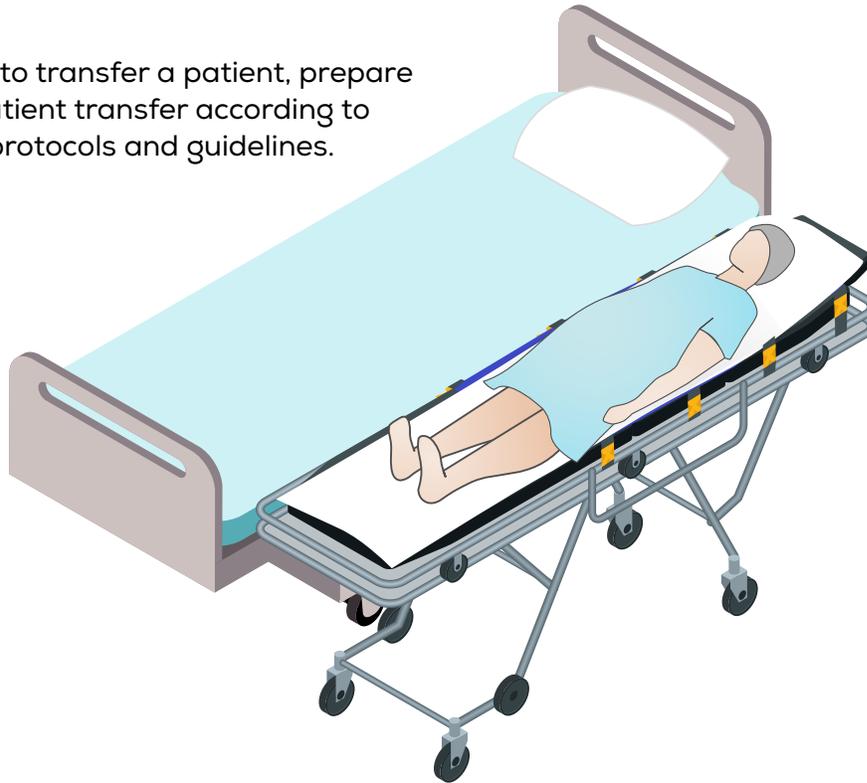


2. Release the Binder Transfer slide sheet from the anchors along the outside edge of the stretcher.

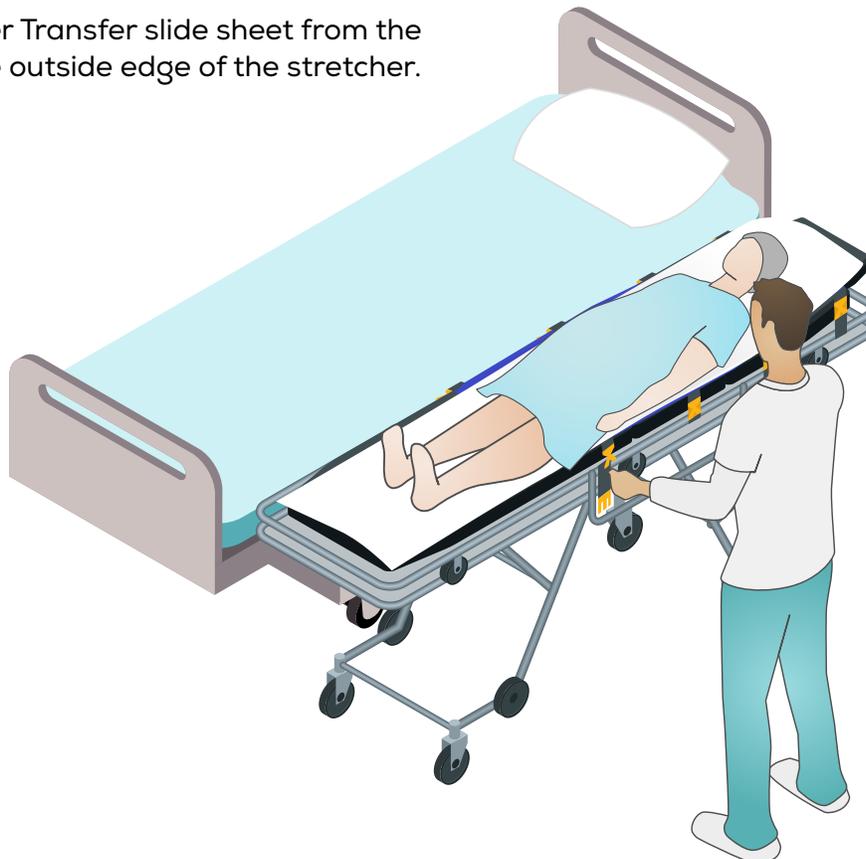

BINDER TRANSFER DEPLOYMENT

3. Gently glide the patient over to the adjacent surface by pulling on the stretcher sheet, NOT the Binder Transfer slide sheet.

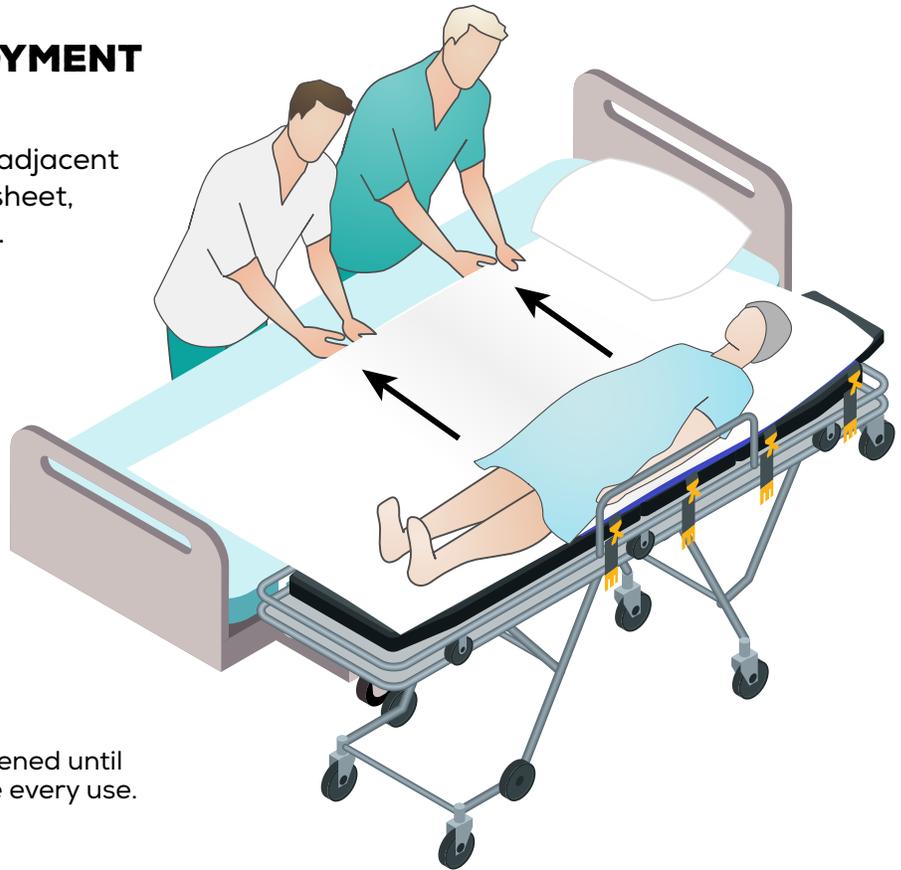

WARNING: Ensure all buckles remain fastened until ready for transfer. Tighten anchors before every use.

NOTE: The Binder Transfer significantly reduces the effort required to transfer a patient. Do not pull with sudden forceful movement and do not lift. This product is not intended for lifting purposes.

CARE AND CLEANING

The Binder Transfer should be inspected and disinfected after every use. Do not use if any piece of the Binder Transfer is frayed, cut, or excessively worn.

CLEANING

Slide sheet: Remove the anchors from the slide sheet before washing. Hand wash with mild detergent according to manufacturer instructions, or machine wash in detergent and water up to 140 degrees

Fahrenheit (60 degrees Celsius) on a gentle cycle, or wipe with disinfectant cleaner (e.g., Cavi Wipes, Opti-Cide MAX, PDIs Super Sani-Cloth). Do not bleach. Hang or wipe to dry. The slide sheet can be machine washed approximately 15 times.

Anchors: Hand wash with detergent and water, or wipe with disinfectant cleaner, e.g., Cavi Wipes, Opti-Cide MAX, PDIs Super Sani-Cloth. A 5% non-chlorinated bleach solution may be used. Hang or wipe to dry.

CALL OR EMAIL US WITH YOUR QUESTIONS AND COMMENTS. WE'RE HERE TO HELP

855-239-5438 | info@phaseintl.com | PHASEINTL.COM